



Activator Instructions for Use

Intended use

The Activator is a complementary device to the MED (Miniaturized Electromagnetic Device).

The device is intended to switch between the 2 operation modes of the MED, and activate the MED device before use.

Introduction

The Activator switches between the 2 operation modes of the MED:

Sleep mode – in this mode the MED does not produce an electromagnetic field, and has a shelf life of two years. You **cannot** use the MED on a patient in this mode.

Active mode – in this mode the MED produces an electromagnetic field. You **can** use the MED on a patient **only** in this mode.

Indications

The Activator is indicated for activation of any MED device.

Note

Read the MED instruction for use before using the Activator.

Caution



The device is supplied non sterile.

Do not sterilize the Activator by autoclave or any other method.

Manufacturing date



The product sticker (at the bottom of the product) indicates the manufacturing date.

Point of Use

The Health Care institution is the Point of Use.

Parts and functions



1. The Activation socket is used to place the MED, while still in the inner tube, for activation/ deactivation.
2. The Activation button is used to switch the MED between the 2 modes:
 - a. Sleep mode: The activation button will light in red for 15 seconds to indicate that the MED placed in the Activation socket is in sleep mode.
 - b. Active mode: The activation button will light in blue for 15 seconds to indicate that the MED placed in the Activation socket is in active mode.
3. Low battery: Low battery light will blink in red to indicate it is time to change batteries. Use two 1.5V AAA size batteries.



Activator Instructions for Use

First use

1. Open the Activator box, pull the upper holder and Remove the Activator from its wrappings
2. Insert two 1.5V AAA size batteries into the battery house on the bottom of the Activator
2. Press the Activation button for 3 seconds – the activation button will blink in red for 2 seconds during the deactivation process, and you will hear a long beeping sound.
3. After the deactivation process is over the activation button will light in red to indicate the MED in sleep mode.

Activation

1. Place the MED , while still in the inner tube, in the Activation socket – The activation button will light in red.

! The MED is to be taken out of the inner tube after the activation, and just before screwing it to the implant.

2. Press the Activation button– The activation button will blink in blue for 2 seconds during the activation process, and you will hear a beeping sound.
3. After the activation process is over the activation button will light in blue to indicate the MED is generating an electromagnetic field and is ready for use in patients.
4. Remove the MED (still in the inner tube) from the Activation socket and follow the MED instruction for use.

Deactivation

1. Place the MED with the inner tube in the Activation socket – The activation button will light in blue.

Storage

1. Store the device at room temperature.
2. Do not bring in contact with water or liquids.
3. Handle with care.

Contact Information:

Please contact MagDent Ltd or its Authorized Representative if you wish to submit a complaint regarding the safety or performance of the device.



Company Headquarter

MagDent Ltd.
Tsivonit St. 1, P.O. Box 89,
14101460352 Tiberias, Israel
Tel: +972.3.643.8585
Fax: +972.153.5428.01876
Email: support@magdentmed.com



European Authorized Representative Office

MedNet GmbH, Borkstrasse 10,
48163 Muenster, Germany



Safe decommissioning
according to local regulations.

Visit us at



www.magdentmed.com

